VOCO

Clip Flow

Instructions for use

GB

CE 0482

DE

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FR

Mode d'emploi

Product description:

Clip Flow is a flowable, light-curing material for temporary fillings, sealings and treatments. Thanks to its elastic consistency, the material is easily removable, also in case of undercuts. Therefore, post-treatment of the cavity is not necessary.

Indications:

- [A] Temporary inlay and onlay treatments of the cavity
- [B] Sealing of openings for implant screws
- [C] Relining material for temporary crowns and bridges
- [D] Block-out material for retentive areas in the dental arch, e.g. before taking impressions
- [E] Covering of the gingival margin
- [F] Fixing of resin matrix during filling placement
- [G] Temporary filling of cavities and sealing of root canal orifices

Preparation:

If necessary, carry out preparation, lining and impression. Adhesion to dental resins or linings based on (meth)acrylat may occur. In such cases apply a suitable insulating agent (e.g. glycerine, vaseline). Avoid the use of lining materials containing eugenol since these could impede the setting of **Clip Flow**. Rinse and dry the cavity before application of **Clip Flow**.

Application:

Apply **Clip Flow** directly from the NDT® syringe using the respective cannula (type 46). In order to ensure the functioning of the non-dripping syringe do not pull back the plunger during or after use.

If necessary, spread **Clip Flow** with an adequate instrument (e.g. probe). Light-cure with a polymerisation light (minimum 500 mW/cm²) a film thickness up to 1 mm for 10 s, (e.g. in case of covering the gingival margin), a thickness up to 5 mm for 20 s and more than 5 mm for 40 s.

Hold the light emission tip of the device as close as possible to the surface of the filling. Otherwise, the curing depth may be reduced.

Finishing:

Excess material can be removed with a scalpel or round bur. Silicone points are suitable for finishing.

Removal of the temporary filling:

Insert an appropriate instrument (e.g. probe or curette) sideways into the temporary filling and extract **Clip Flow** from the cavity.

Warnings/precautions:

- **Clip Flow** contains methacrylates and BHT. **Clip Flow** should not be used in case of known hypersensitivities (allergies) to any of these ingredients.
- In case of eye contact, rinse eyes immediately with plenty of water and seek medical advice.
- The enclosed application cannulae are for single use only and must be discarded after application. Close the syringes after use with the cap provided.
- Phenolic substances, especially preparations containing eugenol or thymol, inhibit the curing of **Clip Flow**. Therefore, do not use zinc oxide eugenol cements or other eugenol-containing substances in combination with **Clip Flow**.

Storage:

Store at temperatures of 4 °C to 23 °C.

To avoid exposure to light and potentially resulting polymerisation, syringes should be closed immediately after dispensing.

Do not use after expiry date.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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VC 60 AA1282 E1 0115 99 © by VOCO
VOCO GmbH