Vinyl polysiloxane bite registration material
DIN 13903

KETTENBACH DENTAL


Simply intelligent

FUTAR®

INSTRUCTIONS FOR USE / Gebrauchsinformation / Mode d'emploi / Gebruiksaanwijzing /
Инструкция по использованию / Istruzioni per l'uso / Instrucciones de uso / Kullanım Kılavuzu /
Instruções de uso / Instrukcja użycia / Інструкція для застосування / 取扱説明書 / تعليمات الاستخدام

CONTENT

Inhalt / Contenu / Inhoud / содержимое /
Contenuto / Contenido / İçerik / Conteúdo /
Treść / Зміст / 目次 / الفهرس

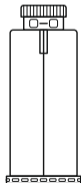Deutsch	4
English	9
Français	14
Nederlands	19
Русский	24
Italiano	30
Español	35
Türkçe	40
Português	45
Polski	51
Українська	58
日本語	65
71	عربي

Vinyl polysiloxane bite registration material
DIN 13903

Futar[®]

Futar[®] Fast

Futar[®] D

Futar[®] D Fast

Futar[®] D Slow

Futar[®] Cut & Trim Fast

For professional use only.

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Product Description

Futar[®], Futar[®] Fast, Futar[®] D, Futar[®] D Fast, Futar[®] D Slow and Futar[®] Cut & Trim Fast are syringeable addition-curing elastomeric bite registration materials intended for making accurate occlusal records.

Indications

All Futar[®] bite registration materials (Futar[®], Futar[®] Fast, Futar[®] D, Futar[®] D Fast, Futar[®] D Slow, Futar[®] Cut & Trim Fast) can be used for making accurate occlusal registrations.

Futar[®] D materials (Futar[®] D, Futar[®] D Fast, Futar[®] D Slow, Futar[®] Cut & Trim Fast) can be used for applications, which require a hard bite registration silicone.

Futar[®] Fast, Futar[®] D Fast and Futar[®] Cut & Trim Fast can be used for “small applications” because of their shorter working time.

Futar[®] D Slow can be used for time-consuming bite registrations because of its longer working time.

Futar[®] Cut & Trim Fast can also be used for an optical registration of occlusal data for CAD/CAM systems without using powder.

Warnings

Do not use Futar[®] bite registration materials for conventional impressions as their high end hardness values severely inhibit withdrawal from the mouth and may result in inadvertent extraction of teeth.

Cautions

Do not use after expiration date.

Do not leave any residual bite registration material in the sulcus or oral cavity.

Do not swallow bite registration material! If swallowed: consult a medical doctor if problems arise or persist.

Avoid contact with the eyes. If there is accidental contact with the eyes: rinse immediately and thoroughly with an eye wash or water. Consult a medical doctor if problems arise or persist.

The product does not normally cause allergic reactions; however, for sensitive persons, an allergic reaction to the product cannot be ruled out.

Use of products containing active sulfur, aluminum chloride or nitrogen compounds (retraction cords containing ferric sulfate, polysulfide impression materials etc.) in conjunction with this

product will interfere with the setting reaction of the vinyl polysiloxane material. Use of these materials requires the area to be rinsed thoroughly to remove all residues.

If gloves are worn when handling the material, they should be tested with a sample of the bite registration material beforehand to ensure compatibility. Do not use latex gloves.

Do not allow the material to enter the sewer or water system to avoid environmental hazard.

Avoid contact with clothing, as the material cannot be removed by dry cleaning.

Note

The total working time and intraoral setting time should be strictly adhered to in order to ensure optimal bite registrations. Should the temperature of the material deviate noticeably from 23°C (74°F), the total working time and setting times will be affected.

Instructions on the Use of Cartridge Material in the Applyfix 4 Dispensing Gun DS-50 1:1/2:1

Insert the cartridge into the Applyfix 4 dispensing gun DS-50 1:1/2:1.

Ensure that the notches on the cartridge base are pointing downwards. The clasp will not close if the cartridge is not properly inserted into the dispensing gun.

The cartridge's tamper-evident closure can now be turned (see Fig. 1).

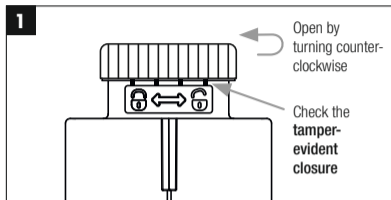
Abb. 1: Opening the tamper-evident closure

The tamper-evident closure ensures that this is an intact original product from Kettenbach Dental.

When used for the first time, the cartridge closure is visibly and audibly separated from the cartridge body. After use, the closure cap can be used to close the cartridge.

Insert the mixing tip with the integrated positioning aid into the coding hole provided for that purpose and rotate clockwise into the thread (see Fig. 2).

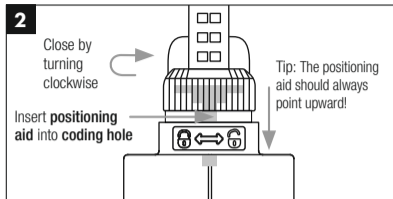
Abb. 2: Inserting the mixing tip

The intuitive handling significantly reduces mistakes when inserting.

Push the plunger into the cartridge and begin applying immediately.

The mixing system ensures even delivery at all times, so it is no longer necessary to throw out the impression material.*

* Does not apply to Futar® D / Futar® D Fast / Futar® D Slow / Futar® Cut & Trim Fast.

Dispense material directly on to the patient's occlusal surfaces or the bite plate.

Replace the closure cap after use. Before using the cartridge again remove the closure cap, attach a new mixing tip and proceed as described.

Cleaning and Disinfection of the Applyfix 4

Remove the cartridge.

Remove any uncured paste using a cloth soaked in alcohol.

The Applyfix 4 dispensing gun can be used several times if properly disinfected. Kettenbach recommends exchanging the gun if there are visible traces of use. Disinfection can be carried out using a commercially available dipping disinfectant. Kettenbach recommends the use of glutaraldehyde-based solutions.

Storage Conditions

Store in a dry place at room temperature; do not expose to sunlight and avoid exposure to extreme temperature fluctuations.

Disinfection

Bite registrations can be disinfected using for example, a 2 % acidic glutaraldehyde solution or use disinfection solutions. See manufacturer instructions.

Disposal

The empty cartridges / mixing tips / applicator tips and set material can be disposed in the household rubbish.

Trademarks

Futar® is a registered trademark of Kettenbach GmbH & Co. KG.

Note

Selected Kettenbach materials/products are available in certain markets only.

For professional use only. Caution: Federal (U.S.A.) Law restricts this device to sale by or on the order of a dentist.

If serious incidents occur that are related to the product, please immediately contact the manufacturer and the competent authority.

Limitation of Liability

Except where prohibited by law, Kettenbach GmbH & Co. KG will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence, or strict liability.

The information provided for Kettenbach products is based on comprehensive research and experience in application technology. Results are furnished to the best of our knowledge, subject to technical changes within the framework of product development. However, users must comply with and consider all recommendations and information in connection with any use.

Updated: October 19, 2021

Product Overview and Technical Data

Product name	Working time at 23°C/74°F ≤	Intraoral setting time at 35°C/95°F ≥	Total setting time* ≥	Hardness (approx.) Shore	Linear dimensional change (approx.) %	Mixing technique	Mixing element
Futar®	30 seconds	1 minute 30 seconds	2 minutes	A 90	- 0.20	Applyfix 4 dispensing gun DS-50 1:1/2:1	Mixing tip, blue, Ø 6.0 mm
Futar® Fast	15 seconds	45 seconds	1 minute	A 90	- 0.20	Applyfix 4 dispensing gun DS-50 1:1/2:1	Mixing tip, blue, Ø 6.0 mm
Futar® D	30 seconds	1 minute 30 seconds	2 minutes	D 43	- 0.10	Applyfix 4 dispensing gun DS-50 1:1/2:1	Mixing tip, blue, Ø 6.0 mm
Futar® D Fast	15 seconds	45 seconds	1 minute	D 43	- 0.10	Applyfix 4 dispensing gun DS-50 1:1/2:1	Mixing tip, blue, Ø 6.0 mm
Futar® D Slow	1 minute 30 seconds	3 minutes	4 minutes 30 seconds	D 43	- 0.10	Applyfix 4 dispensing gun DS-50 1:1/2:1	Mixing tip, blue, Ø 6.0 mm
Futar® Cut & Trim Fast	15 seconds	45 seconds	1 minute	D 35	- 0.20	Applyfix 4 dispensing gun DS-50 1:1/2:1	Mixing tip, red, Ø 4.0 mm

* Total setting time (removal time from mouth) from start of mix

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